dretec

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### Initial Start-Up

#### ● Remove the insulation sheet

- Open the battery door in the back of the scale.
- Remove the insulation sheet.
- Close the battery door and perform zero adjustment.

#### ● Batteries replacement

- Open the battery door in the back of the scale.
  - Remove the used batteries.
  - Insert the batteries (4 x 1.5V AAA) into the battery compartment according to the polarity indications marked inside the compartment.
- ※The digits “8888” will be shown on the LCD.
- Close the battery door and wait until the digits “0.0 kg” are shown on the LCD.

### CAUTION

When the symbol “Lo” appears, the device will power off. Then you shall replace with a new set of batteries. Please replace all four batteries at the same time. Do not mix the old batteries with the new one. Worn batteries are hazardous waste. DO NOT dispose of them together with the household garbage. Please refer to the local ordinances and recycling instructions regarding disposal of the worn batteries. In order to achieve the best performance and protect your scale, please use the authorized/specified batteries.

### Initialising Your Scale

1. Press the platform centre and remove your foot.
2. “0.0 kg” will be displayed.
3. The scale will power off and is now ready for use. This initialisation process must be repeated if the scale is moved. At all other times step straight on the scale.

### Start Measuring

1. Place the scale flatly on a hard surface.
  2. Step onto the platform and stand still until the weight is stable, the numeral and unit on the LCD blink slowly, then the weight is locked. Step quiet off the platform.
  3. The measuring result on the LCD blink for 10 times(about 20 seconds), and then it will turn off automatically.
- ※If the measuring value is unstable for about 15 seconds, for example, the body wobbles, the power will turn off automatically.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:  
(1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Note : This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.